OnTrak TESTSTIK™ 2 COC/THC

50 TesTstiks (100 Tests)

Part No. 1118579

English

Intended Use

OnTrak TesTstik™ 2 for Cocaine/THC is an *in vitro* diagnostic test intended for professional use for the qualitative detection of cocaine metabolite and cannabinoids in urine. The OnTrak TesTstik 2 for Cocaine/THC cutoff levels (300 ng/mL for cocaine and 50 ng/mL for THC) are based on US federal mandatory guidelines.¹ OnTrak TesTstik 2 for Cocaine/THC is not intended for over-the-counter sale.

Samples containing drug below the claimed immunoassay cutoff concentration, but above the GC/MS confirmatory cutoff concentration, consistently render positive results. Refer to the performance section of this package insert for further information.

OnTrak TesTstik 2 for Cocaine/THC provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical test result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmation method.² Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are used.

Summary²⁻⁵

Cocaine, a natural product found in the leaves of the coca plant, is a potent central nervous system (CNS) stimulant and a local anesthetic. Its pharmacological effects are identical to those of the amphetamines (also CNS stimulants), though cocaine has a shorter duration of action. Cocaine induces euphoria, confidence and a sense of increased energy in the user; these psychological effects are accompanied by increased heart rate, dilation of pupils, fever, tremors, and sweating. The “crash” following a cocaine “high” is profound, ranging from irritability, lassitude, and the desire for more drug, to anxiety, hallucinations, and paranoia. Users may resort to other drugs at this time to relieve the depressive effects of the “crash.”

Cocaine is traditionally administered intranasally or smoked in its purer, free-base form. Oral ingestion is ineffective, as cocaine is broken down in the gastrointestinal tract. It is absorbed readily across the mucous membranes of the nose and lungs into the circulation. Its effects are intense but short-lived. Cocaine is rapidly inactivated by hydrolysis of its ester linkages. Blood cholinesterases hydrolyze cocaine to ecgonine methyl ester, while hydrolysis of the parent drug to benzoylecgonine is thought to be nonenzymatic. Both of these metabolites may be further hydrolyzed to ecgonine. Unmetabolized cocaine has an affinity for fatty tissue and rapidly enters the brain. Cocaine metabolites, however, are more water soluble and are readily excreted in the urine along with some portion of unchanged drug. The prominent benzoylecgonine metabolite is the primary urinary marker for detecting cocaine use.

Tolerance had been observed with some chronic, high-dose users. Physical dependence does not appear to occur in abusers, although the development of strong psychological dependence is well known. Cessation of drug use may result in depression, hallucinations, and in extreme cases, psychosis.

The principle psychoactive component of the hemp plant, *Cannabis sativa* (marijuana), is generally accepted to be Δ^9 -tetrahydrocannabinol (Δ^9 -THC), although other cannabinoids may contribute to the psychological and physiological actions of marijuana. The acute effects of marijuana use, concomitant with the desired “high,” are: memory impairment, time confusion, interference with learning, impaired motor skills and depersonalization. These effects are also manifested in chronic users in addition to cardiovascular, pulmonary, and reproductive effects. Marijuana is usually smoked, but alternatively may be ingested, either incorporated into food or as a liquid extract (tea). It is rapidly absorbed from the lungs into the blood with rapid onset of effects; the onset is

slower but prolonged when ingested. The natural cannabinoids and their metabolic products are fat soluble and are stored in the body's fatty tissues, including brain tissue, for prolonged periods after use.

Cannabinoid metabolites are found in blood, bile, feces, and urine and may be detected in urine within hours of exposure. Because of their fat solubility, they also remain in the body's fatty tissues with slow release and subsequent urinary excretion for days, weeks, and even months after the last exposure, depending on the intensity and frequency of use. The prominent Δ^9 -THC metabolite, 11-nor- Δ^9 -THC-9-carboxylic acid (Δ^9 -COOH THC), is the primary urinary marker for detecting marijuana use.

Test Principle

OnTrak TesTstik 2 assays are based on the principles of microparticle capture inhibition. The test relies on the competition between drugs, which may be present in the urine being tested, and drug conjugates immobilized on the membrane for binding to antibody-coated blue microparticles.

When the TesTstik 2 is immersed in the urine sample, some of the sample is absorbed into the TesTstik 2 sample pad. The absorbed sample travels through a reagent strip contained in the device by capillary action. In the reagent strip, the sample rehydrates and mobilizes antibody-coated blue microparticles. The microparticle-urine suspension continues to migrate through the reagent strip and comes in contact with the immobilized drug conjugates. In the absence of drugs in the urine, the antibody-coated microparticles bind to the drug conjugates and blue bands are formed in the result windows.

When drugs are present in the specimen, they bind to the respective antibody-coated microparticles. If sufficient drugs are present, the microparticles are inhibited from binding the appropriate drug conjugate and no blue band is formed in the result window between the brackets for that drug. A presumptive positive (“non-negative”) specimen causes the membrane to remain white.

An additional antibody/antigen reaction occurs at the “TEST VALID” area. The “TEST VALID” blue band forms when anti-bovine serum albumin (anti-BSA) antibodies, which are imbedded in the reagent membrane, bind to the BSA antigen on the blue microparticles. The presence of the “TEST VALID” band indicates that the test has completed, the reagents are viable, and the results are ready to interpret.

Reagents

Each OnTrak TesTstik 2 for Cocaine/THC contains:

1. Azo-blue-dyed microparticles coated with mouse monoclonal anti-benzoylecgonine antibody or anti-cannabinoid antibody and BSA, in a buffered solution containing preservative and dried onto a membrane.
2. Drug conjugates immobilized on membrane.
3. Mouse monoclonal anti-BSA antibody immobilized on membrane.

Precautions and Warnings

For *in vitro* diagnostic use. Exercise the normal precautions for handling all laboratory reagents. Disposal of all waste material should be in accordance with local guidelines.

Storage and Stability

OnTrak TesTstik 2 should be stored at room temperature, 15 - 30 °C (59 - 86 °F), in the original foil pouch. If the foil pouch containing the TesTstik is damaged (e.g., a hole or tear), do not use that TesTstik. Do not use the TesTstik after the expiration date printed on the foil pouch.

Specimen Collection and Preparation

OnTrak TesTstik 2 is formulated for use with urine specimens. Fresh urine specimens do not require any special handling or pretreatment. No additives or preservatives are required. It is recommended that specimens be stored refrigerated (2 - 8 °C or 36 - 46 °F) and tested within two days of collection. For prolonged storage, freezing is recommended.

Specimens containing human-sourced materials should be handled as if potentially infectious. Use safe laboratory procedures such as those outlined in *Biosafety in Microbiological and Biomedical Laboratories* (HHS Publication Number [CDC] 93-8395).

Testing Procedure

Materials provided

50 individually packaged OnTrak TesTstik 2 for Cocaine/THC sticks

Ancillary materials

	Part No.	Quantity
Evidence Tape	42212	100 strips/pkg.
Latex Gloves	42210	50 pairs
Collection Cups	47123	100 cups/box
Adulteration Strips	3111407	25 strips/box

Contact Customer Technical Support for information regarding commercially available control material.

Assay

OnTrak TesTstik 2 for Cocaine/THC is ready for use directly from the foil pouch and must be used within eight hours after removal from the pouch. This time frame may be shorter in high humidity climates. Do not open the pouch unless it is at room temperature (15 - 30 °C or 59 - 86 °F). Specimens should be between room temperature and body temperature 15 - 37 °C (59 - 99 °F). If the specimen was refrigerated, allow it to reach room temperature before testing. To ensure a sufficient quantity of sample for confirmation testing (as necessary), collect a minimum of 30 mL of sample.

1. To expose the sample pad, pull the protective slide back until it stops (up to the "Dip" line).
2. Gently immerse the exposed sample pad into the specimen until the specimen reaches the dip line. Do not immerse the entire device into the specimen. Keep the OnTrak TesTstik 2 in the specimen for approximately 10 seconds.
3. Remove the OnTrak TesTstik 2 from the specimen and while holding the OnTrak TesTstik 2 over the specimen, push the protective slide back over the sample pad until it stops. Allow the test to proceed until a blue band is observed in the "TEST VALID" window (see Results section below). The OnTrak TesTstik 2 may be placed down or held while the test is developing. There is no required timing step. Note: If the Test Valid bands do not appear, re-dip the OnTrak TesTstik 2 for approximately 10 seconds. Allow to sit undisturbed for an additional 1-3 minutes. Once the Test Valid band forms for each assay, proceed to step 4.
4. Remove the tab that covers the result windows from the protective slide. Read and record the results.

Quality Control

Quality control testing at regular intervals is good laboratory practice and may be required by federal, state, or local guidelines. Always check with the appropriate licensing or accrediting bodies to ensure your quality control program meets the established standards.

Internal control: Each OnTrak TesTstik 2 has built-in process controls. The "TEST VALID" band is an internal POSITIVE process control. A blue "TEST VALID" band should always appear if an adequate sample volume is used, the sample and reagent are wicking on the membrane, and the reagents in the "TEST VALID" band and the conjugate-color indicator are working. In addition, the background in the result window should become clear and provide a distinct result. If smears of color appear in the background, nonspecific binding of the antibody may be occurring, making the test result difficult to interpret. The clearing of the result window background may be considered an internal NEGATIVE process control. If the "TEST VALID" band does not appear in the test window, or the background does not clear, then the test is not valid and a new test must be performed.

External control: The use of external positive and negative controls is recommended to test each shipment of product or when a new lot is used. Contact Customer Technical Support for information regarding commercially available control material. The controls should produce the expected results. If expected results are not obtained, do not run test specimens, and contact Customer Technical Support. Follow the appropriate federal, state and local guidelines when running external controls.

Results

Once a full, partial or broken blue band is formed in the "TEST VALID" window (in approximately 3 minutes; see photo #4) remove the cover tab and interpret results. Results may be interpreted for up to 30 minutes after the "TEST VALID" band forms.

A negative result is the presence of a blue band in the test result window between the brackets "[]" next to the respective drug name. (See photo #5 which shows negative results for both cocaine and THC.) The intensity of the blue bands in the "TEST VALID" windows and the result windows may vary between the different tests on OnTrak TesTstik 2 for Cocaine/THC. Since negative specimens may give faint or incomplete color bands, **any** full, partial or broken band in the result window between the brackets "[]" next to the respective drug name indicates a negative result.

A presumptive positive result (drug present at or above the cutoff) is the lack of a blue band in the result window between the brackets "[]" next to the respective drug name. (See photo #6 which shows a positive result for THC and a negative result for cocaine.)

Limitations

See *Specific performance characteristics* section of this package insert for information on substances tested for cross-reactivity in this assay. There is the possibility that other substances and/or factors may interfere with the test and cause erroneous results (e.g., technical or procedural errors). Adulteration of the sample with oxidizing agents and/or acidic or alkaline agents can cause erroneous results. If adulteration is suspected, another sample should be collected and a new OnTrak TesTstik 2 used. For information on how adulterants may affect the performance of this assay, contact Customer Technical Support.

Positive results with these assays indicate the presence of the respective drug or drug metabolites in urine but does not reflect the degree of intoxication.

It has been reported that positive THC test results may be obtained from a nonuser after exposure to extremely high concentrations of marijuana smoke in small, unventilated areas. These extreme exposure conditions are not typical of the usual situations in which the drug is used.⁶ More recent reports indicate that urine cannabinoid concentrations resulting from passive inhalation are not likely to exceed 20 ng/mL.⁷

Specific Performance Characteristics⁸

Accuracy

OnTrak TesTstik 2 for Cocaine/THC was evaluated using clinical specimens screened by an automated immunoassay and confirmed positive by GC/MS at 150 ng/mL for cocaine and 15 ng/mL for THC. Seventy specimens positive for cocaine and 70 specimens positive for THC were evaluated with OnTrak TesTstik 2.

One hundred urine specimens were obtained from a clinical laboratory and screened negative by automated immunoassays relative to a 300 ng/mL cutoff for cocaine and a 50 ng/mL cutoff for THC. A portion (15%) of the negative specimens were also confirmed negative for cocaine and THC by GC/MS at a 150 ng/mL cutoff for cocaine and a 15 ng/mL cutoff for THC. Clinical correlation for all positive and negative specimens is shown in the tables below.

TesTstik 2 Cocaine Assay Clinical Correlation (Cutoff = 300 ng/mL)

	OnLine Negative	GC/MS positive (ng/mL)			Correlation	
		Near Cutoff		630- >10,000		
		247-281	321-364			
TesTstik 2 Cocaine	+	0	10	10	50	94%
	-	100	0	0	0	

TesTstik 2 THC Assay Clinical Correlation (Cutoff = 50 ng/mL)

TesTstik 2	OnLine Negative	GC/MS positive (ng/mL)				Correlation
		Near Cutoff		89-1852		
		39-47	52-60			
+	0	9	10	50	95%	
-	100	1	0	0		

Precision

The precision of OnTrak TesTstik 2 for Cocaine/THC was determined by testing 200 replicates on two lots of TesTstik 2, using multianalyte urine standards containing drugs or drug metabolites at various concentrations (2400 tests in total). The results are summarized in the table below.

Results may vary for samples containing drugs ranging from 50% to 125% of their respective immunoassay cutoff concentration. However, all lots are required to minimally perform with a greater than 95% confidence level that negative results will be attained with drugs at 25% (¼ x) of their respective cutoff concentrations, and a greater than 95% confidence level that positive results will be attained with drugs at 150% (1½ x) of their respective cutoff concentrations.

Urine Standard			Precision Results (% Positive)			
X = Immunoassay Cutoff	Cocaine (ng/mL)	THC (ng/mL)	Lot#1		Lot#2	
			COC	THC	COC	THC
0 x	0	0	0%	0%	0%	0%
¼ x	75	12.5	0%	1%	0%	0%
½ x	150	25	5%	11%	65%	9%
¾ x	225	37.5	66%	36%	100%	77%
1¼ x	375	62.5	100%	99%	100%	100%
1½ x	450	75	100%	100%	100%	100%

Specificity

The following structurally similar compounds to benzoylecgonine and 11-nor-Δ⁹-THC-9-carboxylic acid were tested for cross-reactivity with OnTrak TesTstik 2 for Cocaine/THC. Each compound tested was prepared in normal human urine. The results are expressed in terms of the minimum concentration required to give a positive result.

Minimum concentration required to give a positive result

Cocaine-related compounds	Approx. percent cross-reactivity
Cocaine HCl	0.80
Ecgonine HCl	<0.3
Ecgonine methyl ester HCl	<0.3

Minimum concentration required to give a positive result

THC-related compounds	Approx. percent cross-reactivity
8-β-11-dihydroxy-Δ ⁹ THC	20
8-α-hydroxy-Δ ⁹ THC	12.5
11-hydroxy-Δ ⁹ THC	2.5
11-hydroxy-Cannabinol	2.5
Δ ⁹ THC	0.33
Cannabinol	0.20
Chloroquine	0.07
Cannabidiol	<0.05

Cross-Reactivity with Additional drugs

The following compounds were spiked into pooled negative human urine at a concentration of 100,000 ng/mL and evaluated with OnTrak TesTstik 2 for COC/THC. All spiked specimens were found to give negative results for both cocaine and THC, except as noted.

Acetaminophen	Amitriptyline
6-Acetylmorphine	Amobarbital
Acetylsalicylic acid	Amphetamine
Aminopyrine	Ampicillin

Ascorbic acid	Lidocaine
Aspartame	Methylenedioxyamphetamine (MDA)
Atropine	Methylenedioxymethamphetamine (MDMA)
Benzocaine	Melanin
Benzoylecgonine (except cocaine assay)	Meperidine
Benzphetamine	Methadone
Brompheniramine	d/-Methamphetamine
Buprenorphine	Methapyrilene
Butabarbital	Methaqualone
Caffeine	Methylphenidate
Calcium hypochlorite	Methyprylon
Cannabidiol	Morphine 3-glucuronide
Cannabinol (except THC assay)	Morphine sulfate
Chlordiazepoxide	Nalorphine
Chloroquine (except THC assay)	Naloxone
Chlorpheniramine	Naltrexone
Chlorpromazine	Naproxen
Clemastine	Niacinamide
Cocaine (except cocaine assay)	Norcodeine
Codeine	Cyclizine
Cyclizine	Nordiazepam
Desipramine	Norethindrone
Dextromethorphan	/-Norpseudoephedrine
Dextropropoxyphene	Nortriptyline
Diazepam	Ofloxacin
Dihydrocodeine bitartrate	Omeprazole
Dihydromorphine	Oxazepam
8-β-11-dihydroxy-Δ ⁹ -THC (except THC assay)	Oxycodone
Diphenhydramine	Penicillin G
Diphenylhydantoin	Pentobarbital
Dopamine	Pentoxyverin
Doxylamine	Phencyclidine
Ecgonine	β-Phenethylamine
Ecgonine methyl ester	Phenobarbital
Ephedrine	Phenothiazine
Epinephrine	Phentermine
Erythromycin	Phenylbutazone
Estriol	Phenylephrine
Ethyl morphine	Phenylpropanolamine
17-Ethynylestradiol	Phenyltoloxamine
Fenopropfen	Procaine
Furosemide	Prochlorperazine
Gentisic acid	Promethazine
Glutethimide	Pseudoephedrine
Guaiaicol glycerol ether	Quinidine
Hydrochlorothiazide	Quinine
Hydrocodone bitartrate	Rifampin
Hydromorphone	Secobarbital
p-Hydroxyamphetamine	Sulindac
11-hydroxy-Cannabinol (except THC assay)	Δ ⁹ -THC (except THC assay)
8-α-hydroxy-Δ ⁹ -THC (except THC assay)	Δ ⁹ -THC-9-carboxylic acid (except THC assay)
11-hydroxy-Δ ⁹ -THC (except THC assay)	Tetracycline
Ibuprofen	Tetrahydrozoline
Imipramine	Thebaine
Isoproterenol	Thioridazine
Ketamine	Trifluoperazine
LSD	d/-Trihexyphenidyl
Levorphanol	Trimipramine
	Tyramine
	Verapamil
	Zomepirac

References

1. Mandatory Guidelines for Federal Workplace Drug Testing Programs. Federal Register September 30, 1997. Department of Health and Human Services, Substance Abuse and Mental Health Services Administration.
2. Hawks RL, Chiang CN, eds. Urine Testing for Drugs of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73. 1986.
3. Blum K. Handbook of Abusable Drugs. 1st ed. New York, NY: Gardner Press Inc; 1984.
4. O'Brien CP. Drug Addiction and Drug Abuse. In: Hardman JG, Limbird LE, Molinoff PB, Ruddon RW, Goodman Gilman A, eds. Goodman & Gilman's The Pharmacological Basis of Therapeutics. 9th ed, McGraw-Hill Co. Inc. 1996:557-577.
5. Karch SB. Drug Abuse Handbook. CRC Press LLC 1998.
6. Substance Abuse Testing Committee, Critical Issues in Urinalysis of Abused Substances: Report of the Substance Abuse Testing Committee. *Clin Chem*. 1988;34(3):621.
7. Mulé SJ, Lomax P, Gross SJ. Active and Realistic Passive Marijuana Exposure Tested by Three Immunoassays and GC/MS in Urine. *J Anal Toxicol*. 1988;12:113-116.
8. Data on file at Varian, Inc.

1 Gently tear open the foil pouch to remove the stick. Pull the protective slide back to expose the sample pad.


2 Immerse the sample pad into the specimen until the specimen reaches the DIP LINE. Hold the stick in the specimen for approximately 10 seconds.


3 Remove the stick from the specimen. While holding the stick over the specimen cup, push the protective slide back over the sample pad.

Test complete— ready to interpret results Test NOT complete— DO NOT interpret results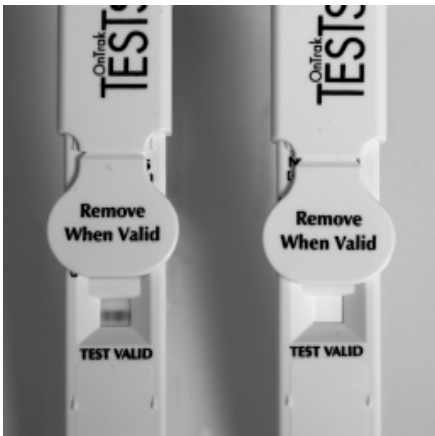

4 Allow the test to proceed until a blue band forms in the "TEST VALID" window (see example on the LEFT). Test is NOT complete until a blue band forms in the "TEST VALID" window (see example on the RIGHT). Once the "TEST VALID" line appears, remove the cover tab and interpret results.

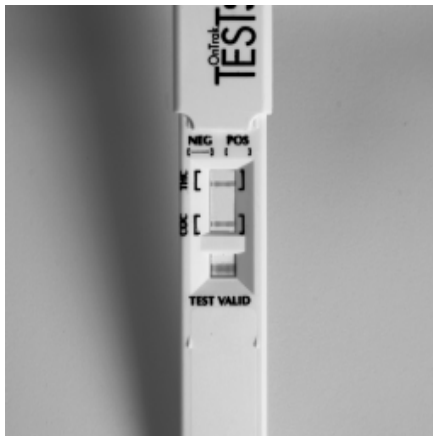5 The presence of a blue band between the brackets indicates a NEGATIVE result. Here cocaine and THC are both NEGATIVE.

6 No color between the brackets indicates a POSITIVE result. Here THC is POSITIVE and cocaine is NEGATIVE.

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Literatur

1. Mandatory Guidelines for Federal Workplace Drug Testing Programs. Federal Register September 30, 1997. Department of Health and Human Services, Substance Abuse and Mental Health Services Administration.
2. Hawks RL, Chiang CN, eds. Urine Testing for Drugs of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73. 1986.
3. Blum K. Handbook of Abusable Drugs. 1st ed. New York, NY: Gardner Press Inc; 1984.
4. O'Brien CP. Drug Addiction and Drug Abuse. In: Hardman JG, Limbird LE, Molinoff PB, Ruddon RW, Goodman Gilman A, eds. Goodman & Gilman's The Pharmacological Basis of Therapeutics. 9th ed, McGraw-Hill Co. Inc. 1996:557-577.
5. Karch SB. Drug Abuse Handbook. CRC Press LLC 1998.
6. Substance Abuse Testing Committee, Critical Issues in Urinalysis of Abused Substances: Report of the Substance Abuse Testing Committee. *Clin Chem.* 1988;34(3):621.
7. Mulé SJ, Lomax P, Gross SJ. Active and Realistic Passive Marijuana Exposure Tested by Three Immunoassays and GC/MS in Urine. *J Anal Toxicol.* 1988;12:113-116.
8. Dokumentation, Varian, Inc.

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Bibliographie

1. Mandatory Guidelines for Federal Workplace Drug Testing Programs. Federal Register September 30, 1997. Department of Health and Human Services, Substance Abuse and Mental Health Services Administration.
2. Hawks RL, Chiang CN, eds. Urine Testing for Drugs of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73. 1986.
3. Blum K. Handbook of Abusable Drugs. 1st ed. New York, NY: Gardner Press Inc; 1984.
4. O'Brien CP. Drug Addiction and Drug Abuse. In: Hardman JG, Limbird LE, Molinoff PB, Ruddon RW, Goodman Gilman A, eds. Goodman & Gilman's The Pharmacological Basis of Therapeutics. 9th ed, McGraw-Hill Co. Inc. 1996:557-577.
5. Karch SB. Drug Abuse Handbook. CRC Press LLC 1998.
6. Substance Abuse Testing Committee, Critical Issues in Urinalysis of Abused Substances: Report of the Substance Abuse Testing Committee. *Clin Chem*. 1988;34(3):621.
7. Mulé SJ, Lomax P, Gross SJ. Active and Realistic Passive Marijuana Exposure Tested by Three Immunoassays and GC/MS in Urine. *J Anal Toxicol*. 1988;12:113-116.
8. Documentation de Varian, Inc.

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1. Mandatory Guidelines for Federal Workplace Drug Testing Programs. Federal Register, 30 de septiembre de 1997. Department of Health and Human Services, Substance Abuse and Mental Health Services Administration.
2. Hawks RL, Chiang CN, eds. Urine Testing for Drugs of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73. 1986.
3. Blum K. Handbook of Abusable Drugs. 1st ed. New York, NY: Gardner Press Inc; 1984.
4. O'Brien CP. Drug Addiction and Drug Abuse. En: Hardman JG, Limbird LE, Molinoff PB, Ruddon RW, Goodman Gilman A, eds. Goodman & Gilman's The Pharmacological Basis of Therapeutics. 9th ed, McGraw-Hill Co. Inc. 1996:557-577.
5. Karch SB. Drug Abuse Handbook. CRC Press LLC 1998.
6. Substance Abuse Testing Committee, Critical Issues in Urinalysis of Abused Substances: Report of the Substance Abuse Testing Committee. *Clin Chem*. 1988;34(3):621.
7. Mulé SJ, Lomax P, Gross SJ. Active and Realistic Passive Marijuana Exposure Tested by Three Immunoassays and GC/MS in Urine. *J Anal Toxicol*. 1988;12:113-116.
8. Datos de archivo, Varian, Inc.

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Riferimenti

1. Mandatory Guidelines for Federal Workplace Drug Testing Programs. Federal Register September 30, 1997. Department of Health and Human Services, Substance Abuse and Mental Health Services Administration.
2. Hawks RL, Chiang CN (a cura di) Urine Testing for Drugs of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73. 1986.
3. Blum K. Handbook of Abusable Drugs. Prima ed. New York, NY: Gardner Press Inc; 1984.
4. O'Brien CP. Drug Addiction and Drug Abuse. In: Hardman JG, Limbird LE, Molinoff PB, Ruddon RW, Goodman Gilman A (a cura di). Goodman & Gilman's The Pharmacological Basis of Therapeutics. 9^a ed., McGraw-Hill Co. Inc. 1996:557-577.
5. Karch SB. Drug Abuse Handbook. CRC Press LLC 1998.
6. Substance Abuse Testing Committee, Critical Issues in Urinalysis of Abused Substances: Report of the Substance Abuse Testing Committee. *Clin Chem.* 1988;34(3):621.
7. Mulé SJ, Lomax P, Gross SJ. Active and Realistic Passive Marijuana Exposure Tested by Three Immunoassays and GC/MS in Urine. *J Anal Toxicol.* 1988;12:113-116.
8. Dati presenti nel file, Varian, Inc.

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Bibliografia

1. Mandatory Guidelines for Federal Workplace Drug Testing Programs. Federal Register September 30, 1997. Department of Health and Human Services, Substance Abuse and Mental Health Services Administration.
2. Hawks RL, Chiang CN, eds. Urine Testing for Drugs of Abuse. National Institute on Drug Abuse (NIDA) Research Monograph 73. 1986.
3. Blum K. Handbook of Abusable Drugs. 1st ed. New York, NY: Gardner Press Inc; 1984.
4. O'Brien CP. Drug Addiction and Drug Abuse. In: Hardman JG, Limbird LE, Molinoff PB, Ruddon RW, Goodman Gilman A, eds. Goodman & Gilman's The Pharmacological Basis of Therapeutics. 9th ed, McGraw-Hill Co. Inc. 1996:557-577.
5. Karch SB. Drug Abuse Handbook. CRC Press LLC 1998.
6. Substance Abuse Testing Committee, Critical Issues in Urinalysis of Abused Substances: Report of The Substance Abuse Testing Committee. *Clin Chem*. 1988;34(3):621.
7. Mulé SJ, Lomax P, Gross SJ. Active and Realistic Passive Marijuana Exposure Tested by Three Immunoassays and GC/MS in Urine. *J Anal Toxicol*. 1988;12:113-116.
8. Ficheiros da Varian, Inc.

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© 2003, Varian, Inc.
Varian B.V.